

REPLACING THE BATTERIES

When the batteries are approximately 25% full the battery symbol will appear on the Qardio App blood pressure measurement screen. Although the QardioArm will continue to measure reliably, you should obtain replacement batteries.

If the empty battery symbol appears on the Qardio App, your QardioArm batteries are depleted and it's time to replace them. You cannot take any further measurements and must replace the batteries.



You should replace all four AAA alkaline batteries at the same time.

Use 4 new, long-life 1.5V, size AAA batteries. Do not use batteries beyond their expiration date, and do not use rechargeable batteries.

To Replace Batteries:


1. Release the battery compartment hatch by pressing the button under the cuff as shown in the drawing.
2. Replace all four AAA alkaline batteries with new ones, ensuring that the polarities are correctly aligned: the + (positive) and – (negative) polarities should match the polarities indicated on the AAA alkaline batteries compartment.
3. Put the battery compartment hatch back in place, pushing until it clicks in place.

4. You will see a green light shining through the battery compartment hatch.

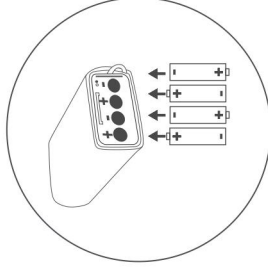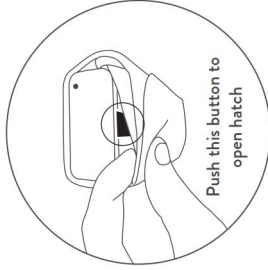


RESETTING THE PAIRING ON iOS

In order to reset the pairing, unwrap the cuff from around the CardioArm to switch on the device and use a paper clip to press the button on the pinhole on the battery compartment hatch. You should see a green light shining through.

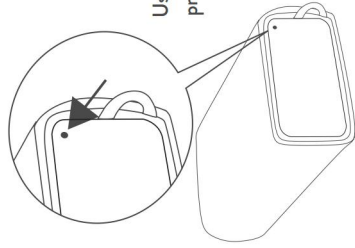


Use a paper clip to
press reset button


If necessary, go into the Settings of your phone or tablet, select the CardioArm and select "Forget this device".

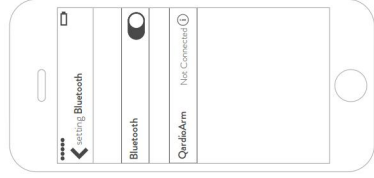

Step 1

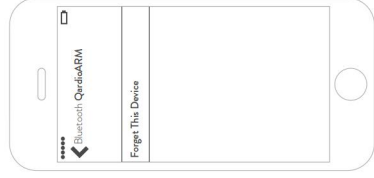

Step 2

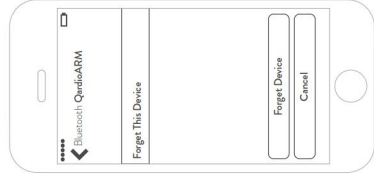

Step 3